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# Proper ways of using TLDs in Diagnostic Radiology

## 1 What is TLD Badge

TLD Badge is a radiation dose measuring device. This enables us to know whether we are working within the safe dose limits prescribed by AERB.

## 3 Wear TLDs below Lead Apron

TLD below apron

TLD above apron

## 5 Change your TLD card every monitoring period

Change your TLD cards every monitoring period (eg. Quarterly) and return used TLD cards to Laboratory for dose assessment.

If TLD fell/left in X-Ray room, and got accidentally exposed, report immediately to the lab and send it for processing.

## Useful Information

### Dose limits of Occupational Exposure (AERB Directive)

Effective Dose 20 mSv in a year averaged over a period of 5 years.

### Annual equivalent dose in the


|                 |         |
|-----------------|---------|
| Lens of the eye | 150 mSv |
| Skin            | 500 mSv |
| Hands and feet  | 500 mSv |

Radiation Warning Placard to be pasted outside Medical X-ray Installation

## 2 Use TLD with Cassette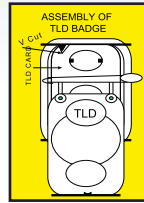

Loading TLD card in the cassette

TLD with cassette

TLD without cassette

## 4 Store TLD- Where ?

Personnel TLD is to be properly stored when not working with radiation

Never store/ leave TLD badges inside X-ray Room/Radiation Area.

- Store control TLD badge in radiation free area all the time.

- Store personnel TLD badge in radiation free area when not in use, along with control TLD badge. (e.g., office room)

Issued by:

### ATOMIC ENERGY REGULATORY BOARD

(Government of India)  
Niyamak Bhavan, Anushaktinagar  
Mumbai-400094.  
www.aerb.gov.in

This poster must be displayed at prominent location for due adherence by all radiations workers.